


## QUICK GUIDE

### Loading Day Cassette (Ref. 1002053)


Loading of Day Cassette should be performed by non-sterile nurse (NSO)

- ① Open the package and remove the Cassette from the tray. The Day Cassette can be used a whole operating day.

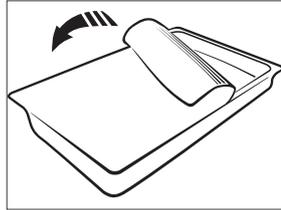


- ② Close all three clamps on the cassette.


- ③ Remove the cap from one of the spikes and connect the spike to a sterile saline bag.

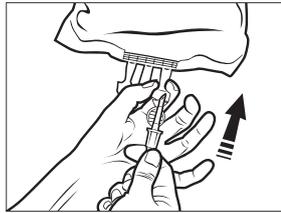


- ④ Push the cassette tubing loop completely over the left pump wheel and push the cassette in place. The cassette should snap in place with a click sound. The yellow LED light beside the Day Cassette button will blink when the Day Cassette is correctly positioned.

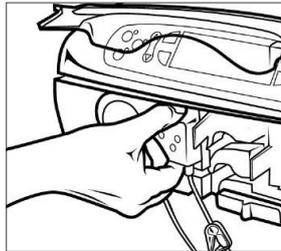

- ⑤ Close the front cover.

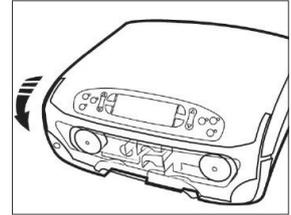

- ⑥ Press the Day Cassette button on the pump. Proper loading is verified in the display.

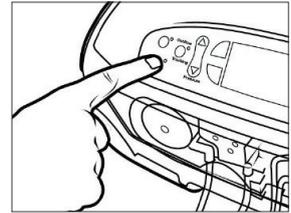


- ⑦ Open the clamp for the primary saline bag


### Loading Irrigation Tubing (Single) (Ref. 1002052)

\*SO: Sterile Operator  
\*NSO: Non-sterile Operator


- ① NSO\*: Open the individual package for either the Double® Pump Single or find the irrigation tubing in the Double® Pump Patient Cassette package.

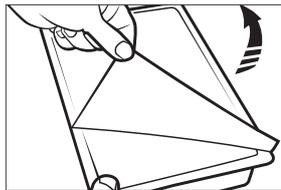

SO\*: remove the irrigation tubing from the package.

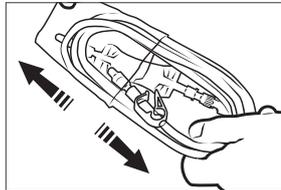


- ② SO: Ensure that the transparent luer lock connection between the tubing and the valve is tightened.

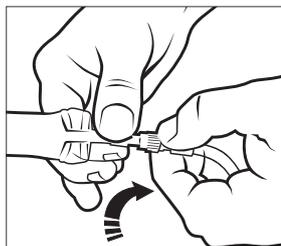

- ③ SO: Close the red clamp on the irrigation tubing and hand over the proximal end (with blue cap) of the tubing to the NSO.

- ④ NSO: If present, remove any used one-way valve from the Day Cassette and connect the blue swivel connector to the transparent female luer lock connector on the distal end of the Day Cassette.

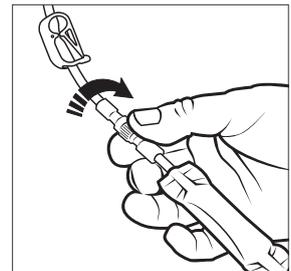

- ⑤ NSO: Connect the distal end of the Irrigation tube to the scope sheath.

- ⑥ NSO: Open the white clamp near the Day Cassette and the red clamp on the irrigation tubing.


- ⑦ The Day Cassette and irrigation tubing can now be filled with sterile saline. When the tube is filled, the red clamp on the irrigation tubing can be used to open or close the flow.

## QUICK GUIDE

### Loading Patient Cassette (Ref. 1002059)


\*SO: Sterile Operator  
\*NSO: Non-sterile Operator

- 1 NSO\*: Open the individual package as shown. The package consists of an irrigation tubing and a Patient Cassette. SO\*: The irrigation tubing is loaded as described on other side.

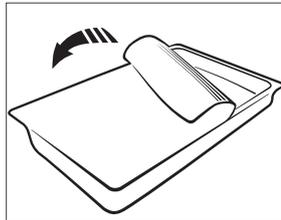


- 5 NSO: Connect the tubing marked WASTE to the waste collection container.

- 6 NSO: Close the front cover.

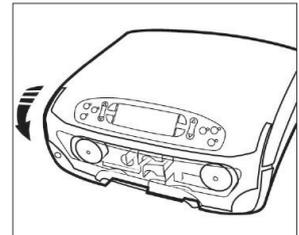


- 2 SO\*: Remove the Patient Cassette from the package. Keep the Cannula and Shaver tubing, and hand the cassette to the NSO to load into the pump.

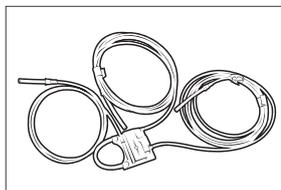


- 7 NSO: Push the Patient Cassette button on the pump. Proper loading is verified in the Display.


- 3 NSO: Push the cassette tubing loop completely over pump wheel to the right. The cassette should snap in place with a click sound. The yellow LED light adjacent to the Patient Cassette button will blink when the Patient Cassette is correctly positioned in the pump.

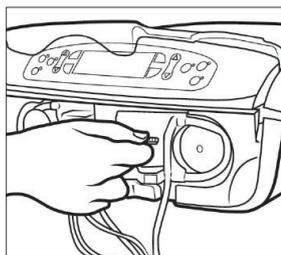

- 8 SO: Connect the end of the OUTFLOW CANNULA tubing to the cannula. The tubing marked SHAVER is connected to the suction port of the shaver hand piece. If a shaver is not used, the tubing can remain disconnected.

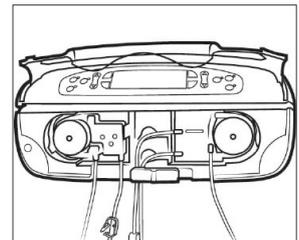

- 4 NSO: Push the three tubings (marked WASTE, SHAVER and OUTFLOW CANNULA) all the way into their respective slots on the pump. Align the tubing together where they exit the slot and hold them in position by closing the slide lock.

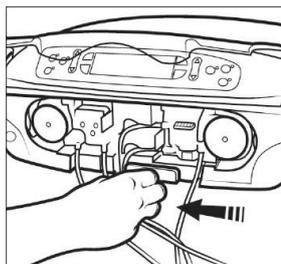

Double® Pump Cassettes		
Item number	Description	Packaging
1002053	Day Cassette	20 pcs/Box
1002059	Patient Cassette	20 pcs/Box
1002052	Single	100 pcs/Box

6000002 Rev.01 2011-10



Medical Vision AB  
Hästholsvägen 32  
SE-131 30 Nacka  
Sweden

Phone: +46 (0)8 452 49 50  
Fax: +46 (0)8 452 49 59  
www.medicalvision.se  
info@medicalvision.se



0413

Medical Vision®  
Technology at your service